


## SWIFT MASK WEARER INSTRUCTIONS

### N95 STYLE MODEL NM-01

Remove Mask From Package.  
Make Sure the Nose Clip Is  
Facing Upward.

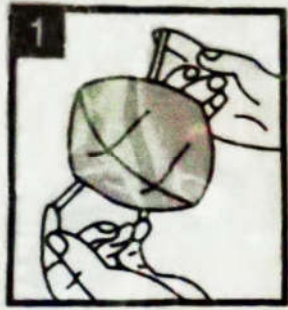

Position The Mask Under  
Your Chin with the  
Nosepiece Up.



While Holding The Mask  
In Place, Pull the Elastic  
Ear Loop Strap To Your  
Ear and Adjust To A  
Comfortable Position.



Using Both Hands, Press the  
Nosepiece to the Shape of  
Your Nose By Pushing  
Downward and Outward  
While Moving Your  
Fingertips Down Both Sides  
of the Nosepiece



Perform a Fit Check By Placing  
Both Hands Completely Over  
the Mask. Be Careful To Not  
Disturb the Fitted Position and  
Exhale Sharply.

